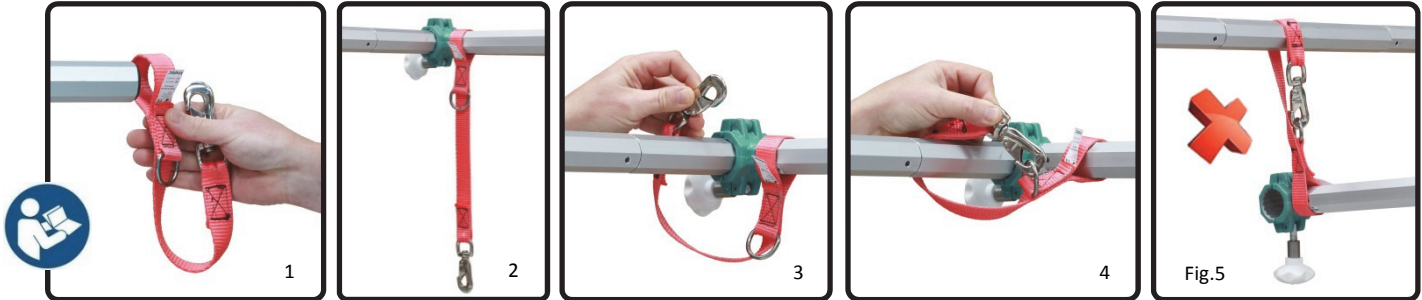
SDS – Sangle de Sécurité • 4655005310

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ENGLISH - Before use, please read this document. Kinetec SAS reserves the right to effect technical modifications.

Safety

Before using Safety Strap, it is essential to have a thorough understanding of this manual. This manual contains instructions for general use and guarantees improved safety.

The KINETEC® system is according to NF EN 60601-2-52 - June 2010. Directive 93/42/CEE from June 14 1993.

Failure to do so could result in patient injury or material damage.

Warning: It is recommended to use this Safety Strap on all articulated traction frames.

Warning: Never put a patient in traction if the Safety Strap is strained (Fig.5)

Note: This Safety Strap does not replace the periodic verification of the correct attachment of the jaws (see user manual of the Kinetec® Traction Frame).

Using

The Safety Strap is used to retain the top crossbar of a Kinetec® Traction Frame in the event that it is disconnected (Fig.5), thus preventing the bar from falling onto the bed.

- 1 – Insert the strap on the top crossbar by the end without clamp.
- 2 – Slide the strap to the other end of the high cross bar, clamp side.
- 3 – Surround the upper part of the head porch bar.
- 4 – Attach the hook to the ring.

Cleaning

Use a DISINFECTANT (PROPANOL/ISOPROPANOL or ALDEHYDE-based solution). Spray the disinfectant on the SURFACES (plastic and metal

components). We advise to pay particular attention to the articulations and securing parts like screws and bolts.

Technical specifications

Product: Medical terms class 1

Storage/transport conditions: Room temperature: -40 to 70°C / -40 to 160°F. Relative humidity: up to 90%

Operating conditions: Room temperature: 10 to 40°C / 50 to 105°F.

Relative humidity: up to 80%

Warranty

The KINETEC® warranty is strictly limited to the replacement free of charge or repair in the plant of the component or components found to be defective.

Kinetec SAS guarantees its Safety Strap for 2 years against all defects of manufacture from the date of purchase by the consumer.

Kinetec SAS is the only organisation able to assess the application of the warranty to its systems.

The warranty will be considered null and void if the device has been used abnormally or under conditions of use other than those indicated in the user's manual.

The warranty will also be considered null and void in the event of deterioration or an accident due to negligence, inappropriate surveillance or inappropriate maintenance, or due to transformation of the equipment or an attempt to repair the equipment.