kinetec®

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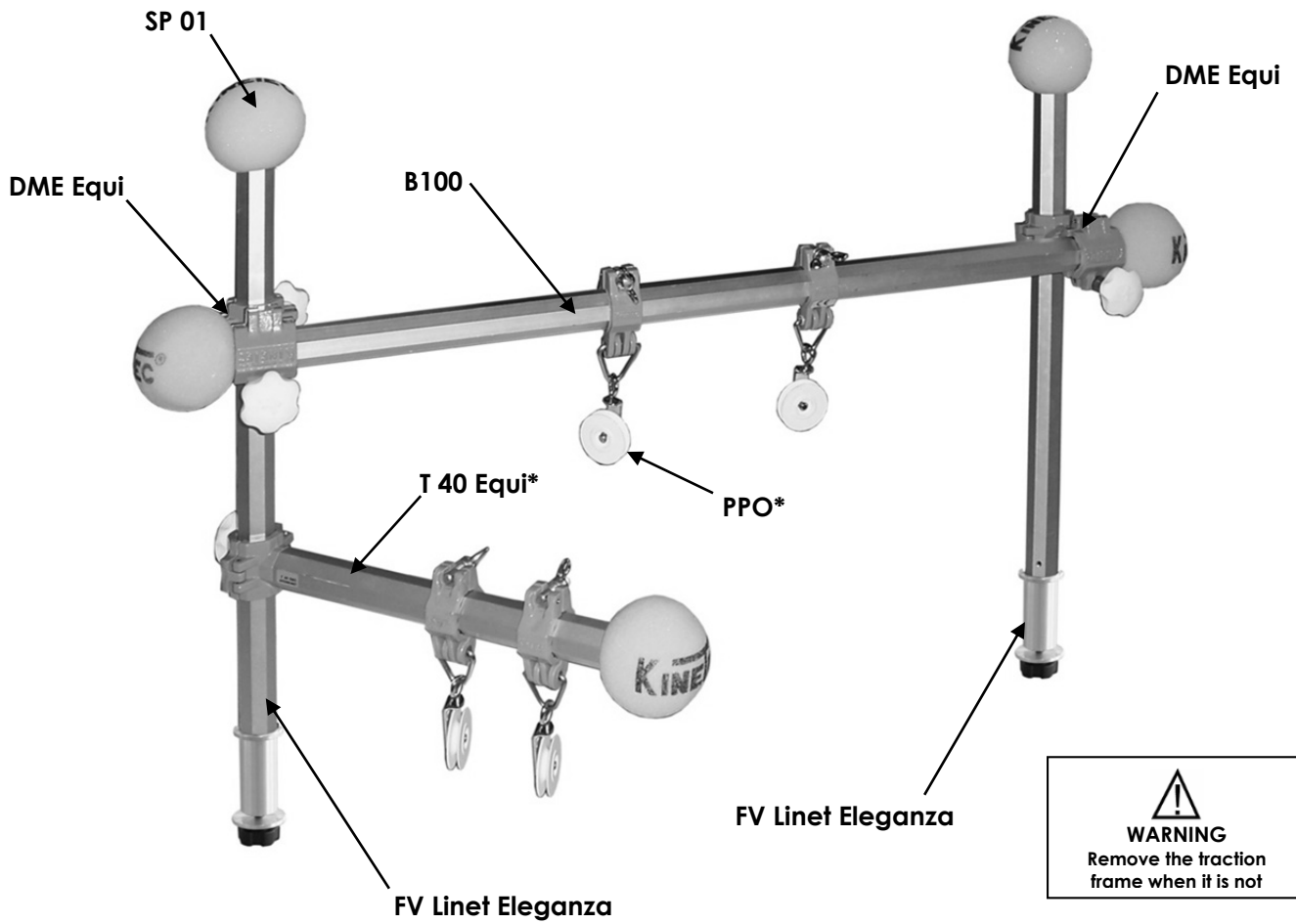
Before use, please read this document.

Safety

- Before using Traction Frame, it is essential to have a thorough understanding of this manual. This manual contains instructions for general use and guarantees improved safety.
- The KINETEC® system is according to NF EN 60601-2-52 norm. Directive 93/42/CEE from June 14 1993.
- The KINETEC® system must not be used with traction frame components from another company.
- Before any use, please make sure that the weight of the patient and of the accessories are compatible with the security load of the bed.
- Check before each use and each week **the fixing of the jaws**, the general state of the accessories and also the connection of the grounding continuity (see picture c).
- The physician must define the protocol and ensure that it is correctly implemented.
- The weights must be positioned outside of the line of the frame ensuring that they do not pose a risk to any person interfacing with the frame.
- Do not position weights above the patient.
- **When the traction frame is in use ensure that frame functions are locked out. All frame functions can be used when the traction frame is no longer in use.**
- **Failure to do so could result in patient injury or material damage.**

Remove the traction frame when it is not used.

Assembly

- Ensure you only use the correct upright bar this will be marked with the "Linet Eleganza, Eleganza 3, Eleganza Smart" label. They mustn't be used with another bed.
- Position the upright bar then fix it with the screws and washers delivered,
- If you remove the foot-board, you create a risk of pinch point; the traction must remain under medical supervision.
- This assembly is not compatible in bed head.

Using

- During the using or during the moving of the bed with the frame, make sure that no components are outside of bed frame (bars, accessories).
- Make sure that during the moving of the bed or during the handling no outer components can knock against the frame (lighting, TV...).
- Do not move the bed with the frame and the patient, but in an emergency situation take care of the obstacles which can provoke weight vibrations (elevator entrance).

Cleaning

- Use a DISINFECTANT (alcohol-free or <5% alcohol solution). Spray the disinfectant on the SURFACES (plastic and metal components). We advise to pay particular attention to the articulations and on holding back zones like the screws and bolts.
- In order to ensure optimal hygiene, you are advised to clean the pads and the frame for each new patient. All the consumables enable hazard-free disposal. We advise also to change the yellow balls (SP01) with each different patient.
- Recommendations to obtain a maximum hygiene of the pads.
Sterilisation of the pads (if necessary) : Sterilised at 134 °C for 18 minutes.
Disinfecting of the pads: Washing at 30°C with use of a disaffection solution during the rinsing cycle. Example of product which can be used: Solution "Baclinge" at 0.125% or "Souplanos" at 0,125% from ANIOS Laboratory. A complete list of distributors in your country is available on request.

Technical specifications

- Product
 - Medical terms class 1
 - Anodising aluminium tubular and epoxy aluminium jaw

- Environment
 - Storage/transport conditions: Temperature: -40 to 70°C / -40 to 160°F
Relative humidity: up to 90%
 - Operating conditions: Room temperature: 10 to 40°C / 50 to 105°F
Relative humidity: up to 80%

Warranty

The KINETEC® warranty is strictly limited to the replacement free of charge or repair in the plant of the component or components found to be defective.

Kinetec SAS guarantees its Traction frames systems for 2 years against all defects of manufacture from the date of purchase by the consumer.

Kinetec SAS is the only organisation able to assess the application of the warranty to its systems.

The warranty will be considered null and void if the device has been used abnormally or under conditions of use other than those indicated in the user's manual.

The warranty will also be considered null and void in the event of deterioration or an accident due to negligence, inappropriate surveillance or inappropriate maintenance, or due to transformation of the equipment or an attempt to repair the equipment.

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