

HOW TO RENT A CPM

ONLINE:

Visit www.kinetecuk.com and click on the red 'Rental' button on the home page. Follow the step-by-step process to order your CPM.

BY PHONE & EMAIL:

Call or email us on the contact details provided on this leaflet.

You will be required to complete a Rental Agreement Form and return it to us. We will collect a credit card payment from you to cover the rental period and a holding deposit. On completion of the payment, we will dispatch the CPM.

DELIVERY & INSTALLATION:

The CPM will be dispatched to you once the payment has been cleared, using a next day service provided by a nationwide courier. Please note, we only deliver on weekdays.

Kinetec do not install the CPM or fit it to the patient. This is the responsibility of the patient who can set it up themselves using the instructions provided. Video set up can be found on YouTube – search 'Kinetec Official'.

COLLECTION:

At the end of the rental period you must make contact with Kinetec to arrange collection of the device or to extend the rental period. Failure to do so may result in additional charges.

CPM CARE:

The patient is responsible for the care of the device during the rental period and will be liable for any repair costs should the device be returned damaged.

Clinically proven to improve patient recovery



For further information on renting a CPM:

- Order your rental CPM online: www.kinetecuk.com
- Call our customer services on 01252 931980
- Email us at enquiries@kinetecuk.com

Kinetec Medical Products UK Ltd
7 Redan Hill Industrial Estate
Redan Road, Aldershot
GU12 4SJ
Tel: 01252 931980
Fax: 01252 329415
enquiries@kinetecuk.com

www.kinetecuk.com

kinetec[®]

CONTINUOUS PASSIVE MOTION

PATIENT RENTAL INFORMATION



WHAT IS CONTINUOUS PASSIVE MOTION (CPM)?

Continuous Passive Motion (CPM) is a post-operative therapeutic treatment that passively – without patient effort – moves a joint through a prescribed range of motion (ROM) for a prescribed period of time. CPM devices are available for all of the major joints – knee, elbow, shoulder, hand, wrist and hip.

TYPICAL USES OF CPM

Orthopaedic surgeons typically prescribe CPM following:

- Total knee replacement
- Ligament reconstruction
- Tendon repair
- Stabilised intra-articular fractures
- Meniscal repair
- Arthroscopic debridement of adhesions

CPM provides significantly better range of motion when used as part of the rehabilitation process

WHY USE CPM?

CPM provides physiological benefits that result in faster recovery and faster return to function. If used post-operatively, CPM can:

- Help prevent joint stiffness and increase ROM
- Promote the healing of the joint surfaces and soft tissue
- Reduces swelling and pain, resulting in improved compliance with physiotherapy and exercise
- Prevent the formation of movement-limiting adhesions



CLINICAL EVIDENCE

There are a broad range of studies and surgeon experiences that support the clinical use of CPM as part of orthopaedic rehabilitation procedures. CPM is best applied immediately post-operatively and continued for up to 6 weeks or as prescribed by the surgeon or physiotherapist.

RENTING A CPM DEVICE

Kinetec provides a full range of CPMs to rent from a period as short as two weeks.

Knee CPM devices typically cost from £10 per day (minimum of 4 weeks) to £12 per day (minimum of 2 weeks) to rent. The cost includes free of charge next day delivery and collection to all mainland UK locations.

All of the CPM units are supplied with manufacturers user instructions and a set of comfortable, disposable, padded leg supports.

COLD COMPRESSION THERAPY

Kinetec are also able to provide cold compression therapy products to help you manage your pain and swelling. Visit our website or call us for details of purchase and rental options for our Cryo Pro and CCT devices.